Product Name	Hwang-toh Agung-yi	Warranty Period
Model Name	DYB-650, DYB-500	One year from the date of purchase
SER NO		
Date of Purchase	20. . . .	
Warranty Period	20. . . .	
Customer Address		
Customer Name		

The Korean Traditional Kitchen

Korean mothers in the olden days spent a lot of their time in a traditional kitchen where wood was burnt in a furnace to cook meals and boil water. Surrounded by wood and the red clay furnace which emit far-infrared rays, these mothers were free from modern-day gynecological ailments. Modern women however suffer from diseases that may not easily be addressed.

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Red Clay Germanium Far-infrared Dry/Wet Heating Seat

User Manual

DYB-650, DYB-500

 **Doo Young**

743-17, Geomsa-dong, Dong-gu Daegu-city Korea
Phone. 82-53-985-6939, 82-70-7718-6939 FAX. 82-53-985-6940

<http://www.dooyoung.or.kr>

About Red Clay...

Our Korean ancestors used red clay to build houses. Red clay is a silicon-type mineral that has long been exposed to solar energy. Because it emits far infra-red ions and has active yeast action, with effective metabolic action against aging and for the prevention of adult diseases, it is beneficial to the human health and is attractive for the modern User who seeks an enjoyable and beneficial sauna experience.

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Contents

- * Product Package / 4
- * To use DYB-650 / 5
- * To use DYB-500 / 9
- * Safety Caution / 12
- * Product Warranty / 14

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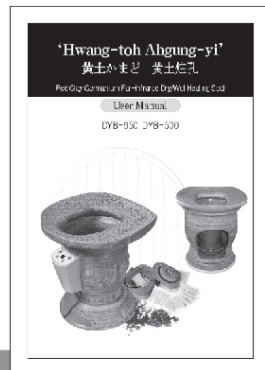
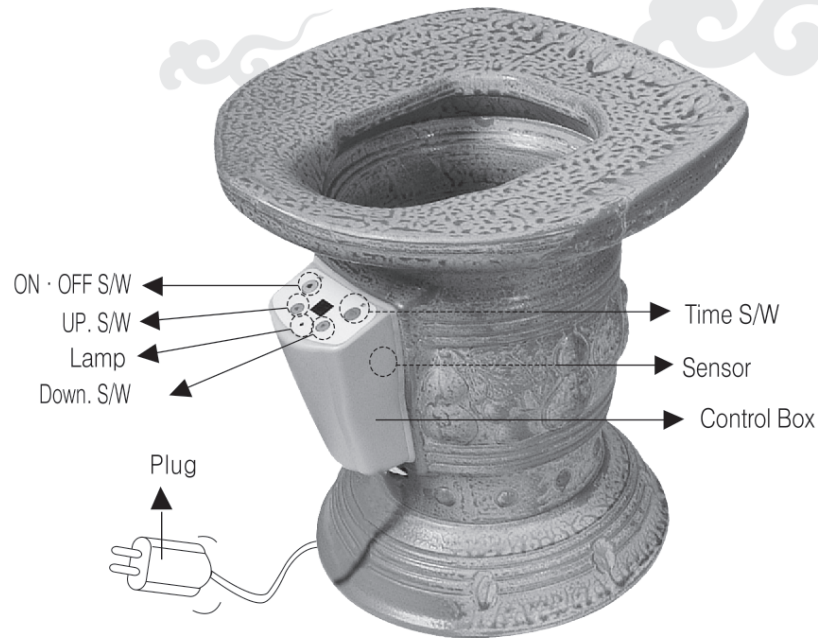
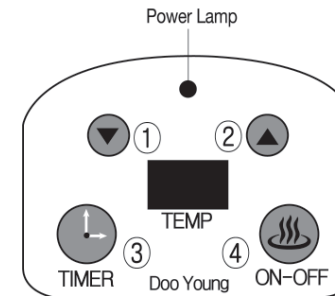
To Use


DYB-650

- ① To lower the temperature
- ② To raise the temperature
- ③ Timer Switch
- ④ Power Switch

1. Plug in and turn on the power.
2. When power is connected the lamp is illuminated.
3. Set the temperature between 10~55℃ and the time for 20~90 minutes.
4. For safety reasons, the product in manual mode will automatically switch off after 2 hours of operation.
5. When the switch is turned ON, the current temperature will be displayed. Raise the temperature by 7~10 ℃. (For example, if the current temperature is 15℃, press the button to set it at 22~25℃.)
6. To use the 'Wet Container', half-fill the wet container with water and a herbal sachet, and close the lid. This will ensure that the herbs are boiled for better effects.
7. To maintain the set temperature, the power will be automatically switched ON and OFF
8. Once the herbs are boiled, open the lid and direct the steam towards your face for a facial massage. Alternatively, release the steam from the container, close the lid, and use the product as a Heating Seat. Ensure that the Seat is cooled before seating.
9. It is not unusual for the temperature of the ceramic material to become hotter than the set temperature.

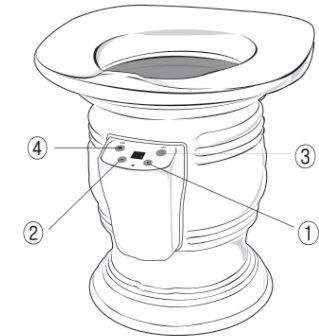To use the Timer : Press button 3 and use the UP or DOWN switches to set the Timer (Times can be set for 20 – 90 minutes using 5-minute intervals)

※ When the switch is turned ON, the current temperature will be displayed.

10. The Dry Mode utilizes far-infrared rays. Use the dry container without inserting a herbal sachet.
11. Herbal sachets can be re-used several times. Ensure that the wet container is half-filled with water each time .
12. If the container becomes contaminated during use, clean the container thoroughly with water.
13. To maintain the set temperature, the power will be automatically switched ON and OFF momentarily during use.
14. Ensure that the container is not over-filled to avoid product damage.
15. Additional herbal sachets can be purchased separately. (20 sachets per pack)
16. Insufficient water in the wet container can be dangerous and may cause smoke to permeate the room and clothes. To prevent this, disconnect the power and allow to cool before using herbs (such as mugwort) and water in the wet container for a heating seat treatment.
17. Use the heating seat regularly for 30 minutes to 1 hour a day for maximum effects.
18. Try herb fumigation spreading both legs open for better result.

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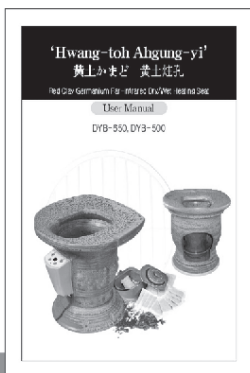
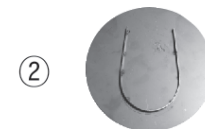
②

①

Standard Heater Assembling Procedure

- Place the Heater Box (Ceramic) Heater Stand (with stainless steel U shape component upside) Heater on the Stainless Steel Stand.
- Insert the Heater wires in the groove and connect with Control Adapter after connecting the wires with Plug Cable.

Operating Procedure (DYB-500)

- Set the desired temperature up using the volume of Control Adapter.
- When desiring to use the fumigation after boiling the herb first, operate the device only after lowering the temperature with the volume.
- Fill the water up with only 1/2 level of Wet Container and pay attention not to have the water overflow out of container.

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DYB-500

- Do not use the product for more than 1 hour at a time.
- If you suffer from diabetes, be cautious of burning from low temperature.
- If you are pregnant, consult with your physician before use.
- If you suffer numbness in the legs, pause the treatment and continue after sufficient time has lapsed.
- Be cautious of burning by high temperature.
- If you suffer from hemorrhoids, DO NOT use the product.
- If you suffer any gynecological disorders, consult with your physician before use.
- Take extra care when the product is used by children or the elderly.
- The temperature display varies when you are seated on the Heating Seat and when you are not. Start on a low temperature.
- Overflow of water during use may cause product malfunction. Ensure that the wet container is not more than half-filled with water.
- Insufficient water in the wet container can be dangerous and may cause smoke to permeate the room and clothes. To prevent this, disconnect the power and allow to cool before using herbs (such as mugwort) and water in the wet container for a heating seat treatment.
- The product is made of ceramic. Take care to avoid impact or breakage. Allow the container to cool before filling with water.
- Check that the power is correctly plugged in. Check if the power supply is functioning.
- Check the fuses.

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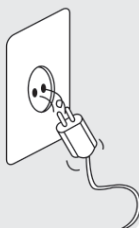
Safety Caution

Remove the power plug from the electric socket if the product is not in use for a long time.

—Continuous connection without use increases the risk of electric shock or fire—

Do not pull the cable to remove the plug from the electric socket. Do not touch the power plug with wet hands as this may cause electric shock or fire.

If water or foreign materials get inside the product, remove the power plug from the electric socket and contact a Service Center near you.

— It may cause electric shock or fire. —

Do not tamper with the buttons on the product as this may cause product breakdown or malfunction.

Do not stand on top of the Heating Seat or place heavy objects on top of it as this may cause product breakdown or malfunction.

If you notice any unusual smell or noise coming from the product, remove the power plug from the electric socket and contact a Service Center near you as this may cause electric shock or fire.

Storage Instructions

- Take care not to drop the product.
- Before cleaning the product, remove the power plug from the electric socket. Use a dry cloth to wipe the product.
- The product is made of ceramic and therefore vulnerable to breakage. Store in a safe place.

※ To replace a faulty cable

- If the power cable is damaged, contact a qualified electrician to replace the cable to avoid risk of electric shock.

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Product Specification

Product Name	Hwang-toh Agung-yi	
Model Name	DYB-650	DYB-500
Manufacture Permit	No. 1687	
KFDA Listing	No. 06-564	
Power Consumption	650W	850W
Nominal Power	110V for Japan, USA	220V for China
Power cable	1.5 meter	
Dimension (mm)	380×380×420	
Weight	13kg	
Manufactured by	Dooyoung	
Date of	20 . . .	
Manufacture A/S Customer Service	T e l : 82-53-985-6939 / 070-7718-6939 FAX : 82-53-985-6940	

1. The Company provides complimentary after-sales services for the warranty period if the Product is found faulty in workmanship during proper use.
2. If found faulty, please contact the Company.
3. In any of the following cases, the Product will be repaired at cost even during the warranty period:
4. Product defects due to User's misuse, abuse or unauthorized repair/modification
5. Product breakdown due to non-compliance with the User's Manual
6. Service outside the warranty period: Your Service Center maintains parts for repair, and will provide services at cost.
7. Please present the warranty card when requesting a service.
8. This warranty shall only apply in Korea.
9. This warranty card shall not be reproduced. Please keep it in a safe place.

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※ About this Product Warranty Card

1. Please fill in the card and have it checked by your retailer. Please keep this card in a safe place.
2. All our products are covered by the Product Liability Insurance in order to protect customer's rights.
3. Service charges (repair cost, parts cost and call-out charges) will apply to the following cases even for services given during the warranty period.

※ Chargeable Services

- Product breakdown due to User's mis-treatment
- Incorrect power supply (e. g. supplying 220V to an 110V-product)
- If the product is damaged or malfunctioning due to unauthorized repair
- Product breakdown caused by natural disasters (such as fire, flood, etc.)
- If the product is damaged or malfunctioning due to modifications by user

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